

## FIBERGLASS CAST APPLICATION

\*Application must be completed by trained veterinarian or vet technician.

### STEP 1: APPLY STOCKINETTE.

Wet the stockinette with clean water, squeeze out excess moisture, and apply on dogs leg from hock to just under hip.

### STEP 2 INSERT BUFFER STRIP.

Cut a tiny slit at the very top of the stockinette. Insert buffer strip through slit, slide down inside stockinette to the paw.

### STEP 3: APPLY FIBERGLASS

Starting at hock (ankle), wrap cast tape upwards around leg, always overlapping 50% of the tape width (. Wrap as high up the leg as possible, then back down, covering evenly.

As it is drying, mold the cast by gently compressing the tape by hand around the knee, forming and bonding the cast tape (3a).

### STEP 4: REMOVE CAST

It is easier to cut and remove the formed cast when it is almost but not completely dry (3 minutes). Use sharp scissors to cut along the buffer strip to carefully remove the cast.

### STEP 5: ALLOW MOLD TO DRY

Maintaining its shape, hold the mold in your hand until completely dry. Carefully tape closed, pack and prepare for shipment back to the Orthopaws offices.

### SHIP TO:

#### TORONTO

200-125 Norfinch Dr,  
North York, ON M3N 1W8

#### MONTREAL

200-125 Norfinch Dr,  
North York, ON M3N 1W8



WATCH OUR  
INSTRUCTIONAL  
VIDEO  
at [www.orthopaws.ca](http://www.orthopaws.ca)

### YOUR BRACING KIT CONTAINS:

- 1 STOCKINETTE
- 1 BUFFER STRIP
- 1 pair GLOVES
- 1 roll FIBERGLASS TAPE

